

PAWPRINT OXYGEN



www.pawprintoxygen.com

What's Inside?

Page 1: Overview

Page 2: Dosage Guidelines

Page 3: RX Approval

Portable Oxygen for Pets

Pawprint Oxygen is the first portable oxygen supply for pets, empowering veterinarians and pet owners to administer life saving oxygen in home, in transport, and in hospital. With lightweight canisters that fit in the palm of your hand, Pawprint Oxygen makes it easy to dispense oxygen for a variety of conditions. Available by prescription only through a licensed veterinarian. Fax prescriptions to 855-699-3366.

How it Works

Each canister provides 10 liters of Oxygen USP which will provide 5-20 minutes of oxygen therapy depending on the flow rate of the regulator. Use the chart on the canister to determine the correct regulator based on the flow rate that was prescribed by your veterinarian.

INSERT the regulator into the canister and TWIST to begin flow. ATTACH the tubing and oxygen mask. When the canister is empty, remove the regulator and **recycle** the canister. If additional oxygen therapy is necessary, attach the regulator to a new canister.



INSERT

TWIST

ATTACH

USE AT HOME

Administer for home rescue to manage respiratory distress resulting from a variety of conditions. Safe to ship and store in your home.

USE IN TRANSPORT

Transport oxygen dependent animals between primary care, referral hospitals, and home. Cost effective and easy to use.

Pawprint Oxygen Canister

Each canister contains 10 liters of Oxygen USP. Safe to ship and store in your home. 2 year shelf life and 100% recyclable.

\$30

Regulator (Choose Flow Rate)

Regulators come in (3) different flow rates: 0.5, 1.0, and 2.0 liters per minute. Your veterinarian will determine the correct flow rate based on your pet's weight. Regulators are reusable, so this is a one-time purchase.


\$65

Pet Oxygen Mask (Choose Size)

Pet Oxygen Masks come in 3 different sizes: Small (for cats), Medium (for small dogs), and Large (for medium and large dogs). Includes 6 ft of tubing. Pet Oxygen Masks are reusable, so this is a one-time purchase.

\$35

If you are a veterinarian or represent an animal hospital, please contact us for wholesale pricing.

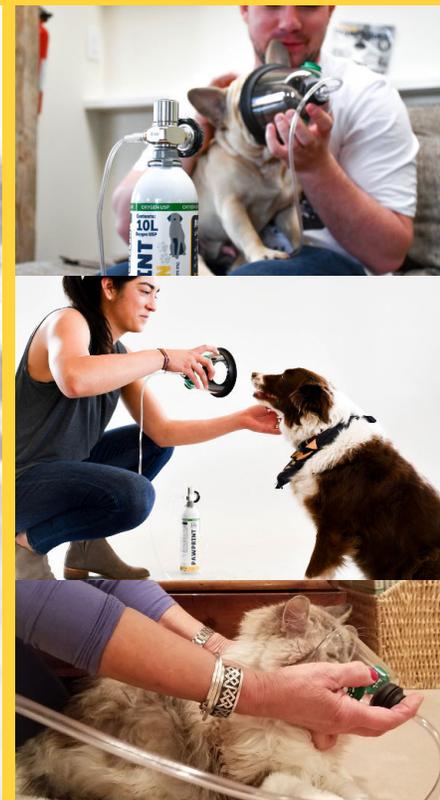

Questions? Call 1-855-OXYGENN today!



For Veterinarians Writing a Prescription

Fill out the Prescription Approval Form on Page 3 of this document

OR

Write your own prescription and include the following information:

- Owner Name, Phone Number, and Email
- Animal Name, Species, Breed, Age, and Weight
- Number of Canister Refills (10 liters of oxygen per canister)
- Regulator Flow Rate (0.5 LPM, 1.0 LPM, 2.0 LPM) see dosage chart below for guidance
- Recommended Size for Pet Oxygen Mask (S, M, L) see below for details

Determining Oxygen Flow Rate

Veterinarians: You are responsible for determining the oxygen flow rate based on the delivery method (i.e. nasal cannula, hood, face mask), animal weight, and desired inspired concentration. A chart is provided based on flow rates and weight to determine the appropriate regulator.

While we provide recommended dosages, we always defer to the attending veterinarian and recommend that dosage and flow rate is decided by the veterinarian. The oxygen regulator will be set to the flow rate that the veterinarian prescribes.

Pet Oxygen Masks come in three sizes:

Small: Best for cats and small mammals

Medium: Best for small dogs with short snouts

Large: Best for dogs over 20 lbs or dogs with larger snouts

Dosage: 50 mL/kg/min

Animal Weight	Regulator	Time Till Empty
<10 lb (4.5 kg)	0.5 LPM	20 min
<20 lb (9.0 kg)	0.5 LPM	20 min
<30 lb (13.6 kg)	1.0 LPM	10 min
<40 lb (18.1 kg)	1.0 LPM	10 min
<50 lb (22.7 kg)	2.0 LPM	5 min

Dosage: 100 mL/kg/min

Animal Weight	Regulator	Time Till Empty
<10 lb (4.5 kg)	0.5 LPM	20 min
<20 lb (9.0 kg)	1.0 LPM	10 min
<30 lb (13.6 kg)	2.0 LPM	5 min
<40 lb (18.1 kg)	2.0 LPM	5 min

Dosage: 200 mL/kg/min

Animal Weight	Regulator	Time Till Empty
<10 lb (4.5 kg)	1.0 LPM	10 min
<20 lb (9.0 kg)	2.0 LPM	5 min

Fax prescriptions to 855-699-3366 or email to info@pawprintoxygen.com

Questions? Call **1-855-OXYGENN** today!



made by Aeronics, Inc.
1080 William Pitt Way
Pittsburgh, PA 15238
1-855-OXYGENN

Prescription Approval for Pawprint Oxygen

Please complete, sign, and return this form via fax (855-699-3366) or email (info@pawprintoxygen.com)
For more information on prescription and dosage guidelines, please visit www.pawprintoxygen.com

Prescription for: Oxygen USP (99.9%), Flow by Mask

Flow Rate (circle one): **0.5 LPM** **1.0 LPM** **2.0 LPM**

Pet Oxygen Mask Size (circle one): **SMALL** **MEDIUM** **LARGE**

Patient Information

Owner Name: _____

Patient Name: _____

Owner Phone: _____

Species/Breed: _____

Owner Address: _____

Patient Weight: _____

Comments: _____

Hospital Information

Hospital Name: _____

Contact Number: _____

Veterinary License Number: _____

Authorizing Doctor: _____

(print)

(sign)

(date)

Fax to 855-699-3366 or email to info@pawprintoxygen.com

Questions? Call 1-855-OXYGENN today!