

10 Liter Oxygen Concentrator

Drive DeVilbiss 1025DS

The Oxygen Concentrator plugs into a wall outlet (120V) and concentrates oxygen from room air, providing high purity oxygen gas (>90%). The Oxygen Concentrator provides an endless supply of continuous flow oxygen therapy and is recommended for pets requiring extended oxygen therapy (more than one hour per day). **Use the Oxygen Concentrator with a Pet Oxygen Mask or a Buster ICU Cage (see back).**

Make: Drive DeVilbiss, Model: 1025DS

Specifications

- Plugs into 120V outlet
- Weighs 42 lbs
- 24.5"x 13.5"x 12"
- Flow rate: 2-10 LPM
- Outlet Pressure: 8.5 psi
- Average Power Consumption: 639 Watts



Questions? Speak with one of our team members by calling **855-699-4366**.
We are available Monday- Friday 9am- 5pm EST.

Getting Started with the Oxygen Concentrator

Drive DeVilbiss - 10 Liter Model

Step 1: Power up the Concentrator

1. Remove the Oxygen Concentrator from the package and plug the power cord into a 120V outlet.
2. Press the power switch to turn the device ON. Oxygen will start flowing immediately.
3. Twist the Dial to adjust the oxygen flow rate based on your prescription.

Tips

- We recommend a flow rate of 10 Liters per Minute for Large Oxygen Cages.
- The Concentrator should be at least 6 inches from a wall to allow the fan to work properly.
- We recommend keeping the Oxygen Cage unzipped at least 3 inches for ventilation.

Step 2: Connect the Concentrator to an Oxygen Cage or Oxygen Mask

Buster ICU Oxygen Cage

- Connect one end of the oxygen tubing to the venturi piece on the Oxygen Cage and connect the other end of the oxygen tubing to the silver oxygen outlet port on the Concentrator.
- Each Oxygen Cage includes: oxygen tubing, venturi set, ice packs, nebulizer piece
- Refer to the Buster ICU Oxygen Cage manual for more in-depth set up instructions.



Available in 3 sizes:

- Small (\$299.99): 18" x 13" x 13"
- Medium: (\$399.99): 24" x 18" x 18"
- Large (\$599.99): 44" x 26" x 26"

NOTE: Large Cages require a 10 Liter Oxygen Concentrator to achieve sufficient oxygen levels.

Pet Oxygen Mask

- Connect one end of the oxygen tubing to the Pet Oxygen Mask and connect the other end of the oxygen tubing to the silver oxygen outlet port on the Concentrator.
- Adjust the oxygen flow rate based on the prescription from your veterinarian.
- Slowly place the Pet Oxygen Mask over the snout of your pet to begin oxygen therapy.



Available in 3 Sizes:

- Small: for cats
- Medium: for small dogs
- Large: for medium and large dogs
- Includes 6ft of oxygen tubing

Oxygen equipment should be used at the direction of your veterinarian.